Sempach

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e 250 ml

REF 18564  
PZN 01264592

**Composition:** 100 g contains Propan-1-ol 50.0 g, Didecyl-dimethylammonium chloride 0.075 g, perfume, excipients  
**Contains: Propan-1-ol**

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**GB FOR MEDICAL DEVICES**

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**PT PARA DISPOSITIVOS MÉDICOS**

Bactericidal, yeasticidal, mycobactericidal, limited virucidal

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**GB** Not compatible with acrylic glass and materials sensitive to alcohol.

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**GB Use:** Suitable for alcohol resistant surfaces of medical devices e.g. examination chairs, hospital beds and theatre equipment. Not for use on invasive devices. Moisten surface with undiluted solution until it is completely wet.

Keep away from heat, hot surfaces, sparks, open flames

and other ignition sources. No smoking. Do not breathe vapours. Wear protective gloves/protective clothing/eye protection/face protection. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Call a POISON CENTER/doctor if you feel unwell. Dispose of this material and its container to hazardous or special waste collection point.

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# Meliseptol® rapid

**Composition:** 100 g contains Propan-1-ol 50.0 g, Didecyl-dimethylammonium chloride 0.075 g, perfume, excipients.

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**Rapid disinfectant**  
for small surfaces / for medical devices

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**GB** Flammable liquid and vapour. Causes serious eye damage. May cause drowsiness or dizziness. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not breathe vapours. Wear protective gloves/protective clothing/eye protection/face protection. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Call a POISON CENTER/doctor if you feel unwell. Dispose of this material and its container to hazardous or special waste collection point.

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**B. Braun Melsungen AG**
**B. Braun Medical AG**