Sempach

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GB Fast acting alcoholic disinfectant

- for small surfaces • for medical devices

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GB Instruction for use: Unscrew the lid of the dispenser. Open the foil bag and adjust the roll of wipes to the shape of the dispenser and push the roll into the dispenser. Pull out the first wipe from the middle of the roll and thread through the star on the lid. Screw lid onto the dispenser. See dispenser for disinfection method of medical devices.

Active in one minute against bacteria, yeasts and some enveloped and non-enveloped viruses.

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For professional use

dermato-
logically tested
simple new
handling

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B | BRAUN

L01

B | BRAUN

B. Braun Melsungen AG
Carl-Braun-Strasse 1
34212 Melsungen, Germany
Tel +49 (0) 5661 710

M B. Braun Medical AG
Seesatz 17
6204 Sempach
Switzerland

Tücher/Lingettes/Tissues/Salviet-
tine/Doekjes/Toallitas/Toalhets

REF 18707 | PZN 01889412

CHZB0942 | N-16096, N-16098

7 612479 080555

€ 0123

87960 - 2018-05-22

Composition: 100 tissues impregnated with 180 g solution, 100 g solution. Propan-1-ol 50 g, Didecylmethylethylammonium chloride 0,075 g, perfume, excipients.

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GB Danger: Flammable liquid and vapour. Causes serious eye damage. May cause drowsiness or dizziness. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Dispose of this material and its container to hazardous or special waste collection point.

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For Safety Phrases and further information, see Dispenser.

Meliseptol® HBV - Tücher / Tissues

Meliseptol® HBV

Tücher / Lingettes / Tissus / Salviettine /
Doekjes / Toallitas / Toalhetes

Spenderbox / Distributeur / Dispenser / Distributore
Dispenserbox / Dispensador / Dispensador

Composition: 100 tissues impregnated with 180 g solution.
100 g solution contains: Propan-1-ol 50 g, Didecylmethyl-
ammonium chloride 0.075 g, perfume, excipients.

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GB **Fast acting alcoholic disinfectant**
for small surfaces / for medical devices

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