Labelingspezifikation

Dokument N°: LS-19796

Version: 08

Gültig ab: System Date

Seite: 1 von 2

Sempach

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GB Rapid disinfectant

- for small surfaces
- for medical devices

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perfume
aldehyde **free**
ready for use

1 active
within
minute

REF 19796
PZN 07314546

e 750 ml

880481 - 2015-12-03

B | BRAUN

CH CHZN1018
DE N-30782, N-30783

CE 0123

Meliseptol® Foam pure

Active against bacteria, TbB, fungi, enveloped viruses
(incl. HBV, HCV, HIV)*, Rotavirus, Polyomavirus, Norovirus
Material compatible with dialysis machines e.g. Dialog+ and infusion systems e.g.
Perfusor® Space / Infusomat® Space.

Composition / Zusammensetzung / Composizione / Bestanddelen
Composição / Composição: Active substances in 100 g solution:
17 g Propan-1-ol, 0.23 g Didecyltrimethylammonium chloride.
Excipients: < 5 % non-ionic surfactants, purified water.
Contains: **Propan-1-ol**

FR Danger. Liquide et vapeurs inflammables. Provoque des lésions oculaires graves. **GB Danger.** Flammable liquid and vapour. Causes serious eye damage. **DE Gefahr.** Flüssigkeit und Dampf entzündbar. Verursacht schwere Augenschäden. **IT Pericolo.** Liquido e vapori infiammabili. Provoca gravi lesioni oculari. **NL Gevaar.** Ontvlambare vloeistof en damp. Veroorzaakt ernstig oogletsel. **ES Peligro.** Líquidos y vapores inflamables. Provoca lesiones oculares graves. **PT Perigo.** Líquido e vapor inflamáveis. Provoca lesões oculares graves.

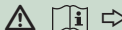
* acc. recommendation of RKI, Bundesgesundheitsblatt 01.2004
** corresponding DGHM 2001 acc. Exner et Gebel 2007, Hyg Med 32/4: 128-129

Exposure times / Temps d'action	
Bacteria, yeasts, enveloped viruses / Bactéries, Levures, Virus enveloppés (incl. HBV, HCV, HIV)*, Rotavirus. DGHM 2001/EN 1040/EN 1276/EN 13727/13624/13697** DWV/RKI	1 min.
Bacteria, yeasts, high organic load / Bactéries, Levures en conditions de saleté DGHM 2001, EN13727/13624/13697**	2 min.
Tuberculosis bacteria (M. terrae) / Mycobactéries EN 14348/EN 14563	3 min.
Polyomavirus DWV/RKI, Norovirus (MNV)	5 min.

B. Braun Melsungen AG
Carl-Braun-Strasse 1
34212 Melsungen, Germany
Tel +49 (0) 5661 710

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GB Use: Also for medical devices with sensitive surfaces e.g. ultrasound devices, infusion pumps, dentist's chairs, artificial leather beds. Coat surfaces completely with product. Not for use on invasive devices. Exposure times: see table.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not breathe vapours. Wear protective gloves/protective clothing/eye protection/face protection. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes, several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Call a POISON CENTER/doctor if you feel unwell. Dispose of this material and its container to hazardous or special waste collection point.

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Meliseptol® Foam pure

Composition: Active substances in 100 g solution: 17 g Propan-1-ol, 0.23 g Didecylidimethylammonium chloride, Excipients: < 5 % non-ionic surfactants, purified water.

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Rapid disinfectant for small surfaces for medical devices

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GB Danger. Flammable liquid and vapour. Causes serious eye damage. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not breathe vapours. Wear protective gloves/protective clothing/eye protection/face protection. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Call a POISON CENTER/doctor if you feel unwell. Dispose of this material and its container to hazardous or special waste collection point.

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B. Braun Medical AG