
# Flexima® / Softima® Active Flexima® / Softima® Active O' convex

---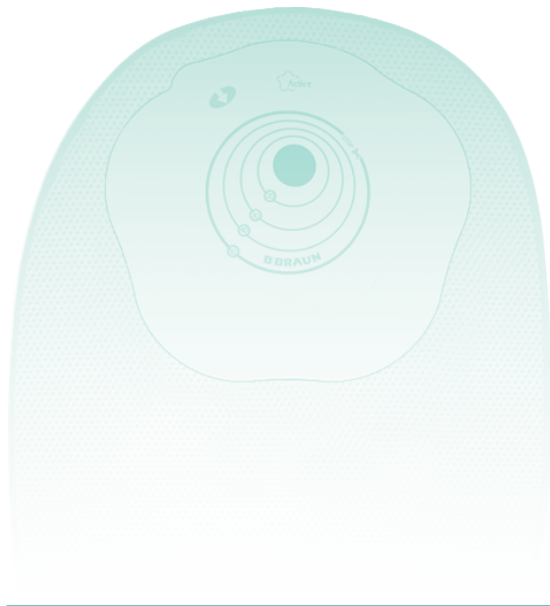

**EN** Instructions for use

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








---





**B | BRAUN**

# Flexima® / Softima® Active

## Flexima® / Softima® Active O' convex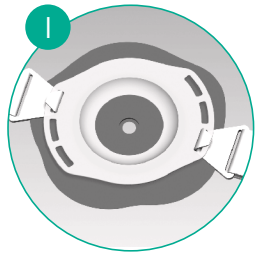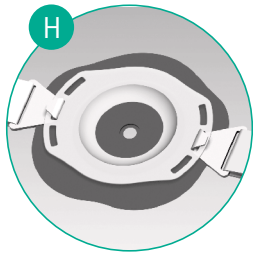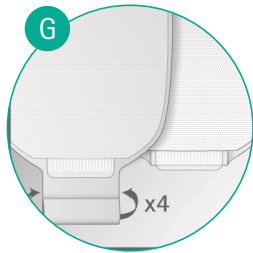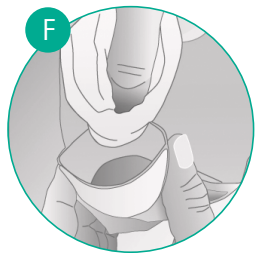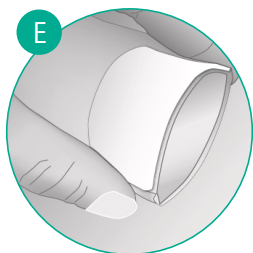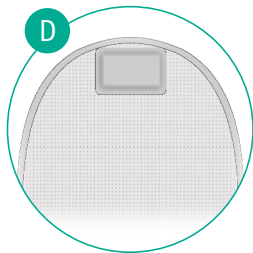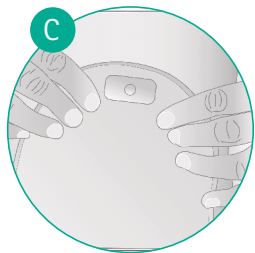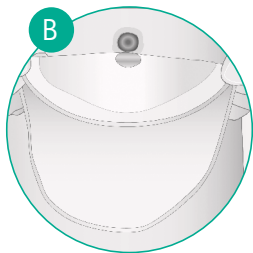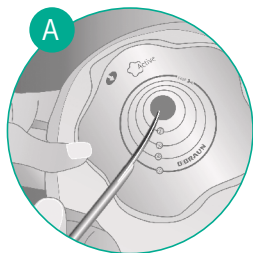

Flexima® / Softima® Active  
Flexima® / Softima® Active O' convex

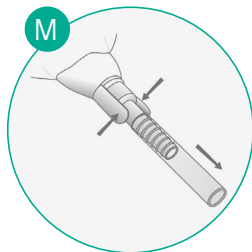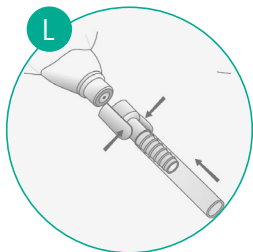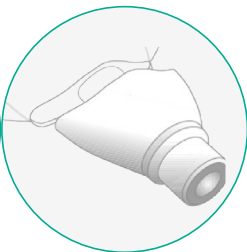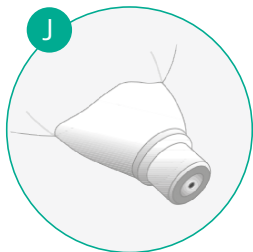# EN Instructions for Use

**Flexima®/ Softima® Active is a one-piece range of collecting pouches for enterostomates. It offers a unique skin protector developed by B. Braun using the B. BraunTech formulation which provides high comfort thanks to its high flexibility and immediate adhesivity.**

Available in\*:

- closed pouch with flat and soft convex skin protectors,
- drainable pouch with an integrated soft clamp called "Roll'up", in flat and soft convex versions
- uro pouches in soft convex version

Indications for the soft convex skin protector:

The Soft Convex skin protector is specifically designed to ensure a secure seal around the stoma:

- for stomas located in light or moderate skin folds
- or flushed or slightly retracted stomas

The soft convex skin protector helps to prevent leakages, by tight-fitting the peristomal area. For extra security you can use one out of the three positions (H,I) of the belt loops of the soft convex skin protector to fix the belt. If you wish to use a belt, choose two notches that are diametrically opposed (H, I).

## 1 Preparation of the skin:

- Wash your hands.
- Gently clean the area around the stoma with lukewarm water and carefully dab the skin with soft tissues.
- Avoid using any disinfectant (either alcohol or any

antiseptic) as this could damage the skin and prevent proper adhesion of the pouch.

## 2 Before setting up your appliance, first:

- Cut the skin protector if you are using a cut-to-fit pouch so it fits the size of your stoma, taking good care to follow the cutting guide print (do not cut beyond the limit). Ensure you leave 2 to 3mm around the stoma (A).

## 3 Fitting your appliance:

- Separate the pouch walls by crumpling it slightly to introduce air into it.
- Peel off the clear release by pulling on the arrow on the tab.
- Position the bottom of the skin protector around the base of the stoma (B).
- Press the bottom part of the skin protector smoothly onto the skin, then place and press onto the upper part (the skin protector is flexible and follows body movements) (C).
- Smooth the entire surface of the skin protector, working from bottom to top, to ensure perfect adhesion.
- In case of error in the positioning, it is possible to remove the pouch and replace it properly.

## 4 Use of drainable pouch Roll'Up:

Closing of the outlet:

- Fold the sleeve four times using the guide and fold the tab back.

- To seal the pouch, apply pressure on all the length of the two self-fastening strips facing each other.

Please note: the closed sleeve is hidden under the soft backing material (G).

#### Emptying the pouch:

- Position yourself to carry out the drainage, release the two self-fastening strips and unroll the sleeve. Be careful, the drainage will start at this point. To fully open the outlet place your fingers on each side and press slightly (E).
- Once the drainage is complete, expel, as much as possible, any matter still in the sleeve by pressing it lightly from top to bottom. Carefully clean its extremity (be careful: use non-shedding material) (F).
- Then close the pouch.

To close the pouch, fold the sleeve as indicated above.

#### **5** Urostomy pouches (emptying):

- With the pouch pointing downward, direct the drain cap into the toilet.
- Gently rotate the cap counterclockwise until you achieve the desired flow rate (K).
- To close the cap when the pouch is empty, return the cap to the original position by turning it clockwise.

#### **6** Removing your appliance:

Pull slightly on the top of the pouch and remove the pouch from the top to the bottom while maintaining gentle pressure on the skin around the stoma with your other hand.

#### **7** Use of the patch cover for filter protection:

If you wish to inactivate the filter before you take a shower or a bath, stick an adhesive patch on the top of the pouch so that the hole for gas evacuation is blocked (D).

#### **8** Use of the connector for the night bag:

- Clip on the connector at the end of the closed cap by pushing it slightly (L). Open the cap turning it half a turn counterclockwise to allow urine to flow in the night bag. The connector can freely rotate to avoid tubing twisting.
- To un-clip, close the cap of the bag by turning it half a turn clockwise. Take the connector by the two wings and pull with caution (M).

The Flexima® urostomy bag can be connected to standard urine bags and to the Urimed® bag 2L.

#### CAUTION:

- Take good care to crumple the pouch slightly when fitting the appliance in order to separate the walls so that urine can flow freely!
- After the complete emptying of the pouch, pay attention to close the outlet completely by aligning it with its basis.

*\* Availability may vary from one country to another.*

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**FR** B. Braun Medical SAS
**GB** B. Braun Medical Ltd.
B. Braun Medical SAS | 204 Avenue du Maréchal Juin | 92100 Boulogne-Billancourt | FRANCE