
- GB** Instructions for use/Technical description
- USA** Endo-clips and forceps
- D** Gebrauchsanweisung/Technische Beschreibung
Endo-Clips und Zangen
- F** Mode d'emploi/Description technique
Pinces pour poser les clamps
- E** Instrucciones de manejo/Descripción técnica
Endoclips y pinzas
- I** Istruzioni per l'uso/Descrizione tecnica
Endo-Clip e pinze
- P** Instruções de utilização/Descrição técnica
Clipes e pinças de endoscopia
- NL** Gebruiksaanwijzing/Technische beschrijving
Endo-clips en tangen
- S** Bruksanvisning/Teknisk beskrivning
Endo-clips och tänger
- RUS** Инструкция по применению/Техническое описание
Эндоклипы и зажимы для наложения
- CZ** Návod k použití/Technický popis
Endoskopické svorky a kleště
- PL** Instrukcja użytkowania/Opis techniczny
Klipsy endoskopowe i kleszcze
- SK** Návod na používanie/Technický opis
Endo-svorky a kliešte
- TR** Kullanım Kılavuzu/Teknik açıklama
Endo klips ve forsepsler

B | BRAUN
SHARING EXPERTISE

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 (0) 7461 95-0 | Fax +49 (0) 7461 95-26 00 | www.aesculap.com

Aesculap – a B. Braun company

TA-Nr. 012009 03/13 V6 Änd.-Nr. 46715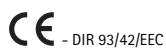

Technical alterations reserved

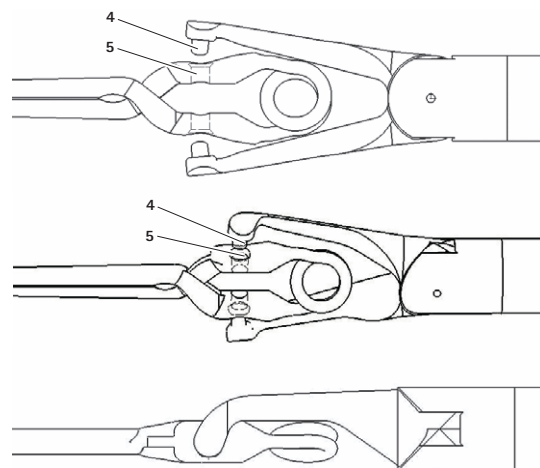
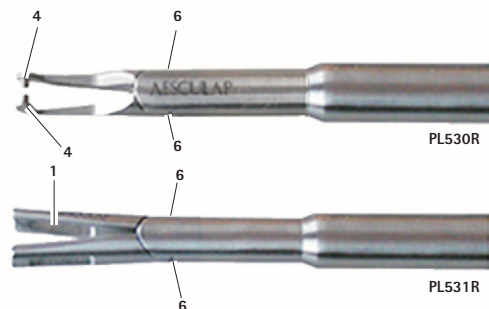
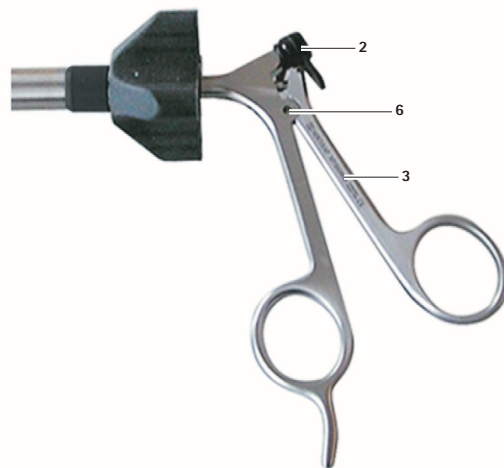
Fig. 1

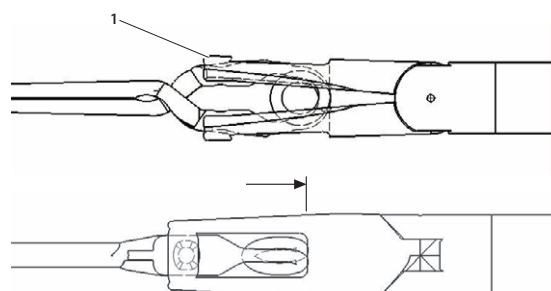
Fig. 2

Aesculap®

Endo-clips and forceps

Legend

- 1 Jaw part
- 2 Luer lock connector for cleaning channel
- 3 Handle
- 4 Nibs
- 5 Holes for clips
- 6 Lubrication points

Symbols on product and packages

Caution, general warning symbol
Caution, see documentation supplied with the product

Intended use

Note
The clips may only be used temporarily during the operative procedure and must only be used with the original Aesculap clip applying/removal forceps PL530R and PL531R.

The clip applying/removal forceps and clips have been designed for endoscopic use. The atraumatic clip applying / removal forceps are intended for applying and removing clips PL541S–PL549S, as well as PL593R and PL594R.

The vascular clips PL542S–PL549S are used to temporarily clamp vessels.:

- Arterial vascular clips
 - without gold-plated spring area and with high closing force
- Venous vascular clips
 - with gold-plated spring area

The intestinal clip PL541S is used to temporarily clamp the intestine. The roughened surface of the inner side of the clamp prevents the intestine from slipping out.

The clips PL593R and PL594R are used in combination with the intracorporeal retractor PL595SU.

Overview:

Art. no.	Designation	Marking
PL544S, PL545S, PL548S, PL549S	Arterial vascular clip	
PL542S, PL543S, PL546S, PL547S	Venous vascular clip	gold-plated spring area
PL541S	Intestinal clip	
PL593R, PL594R	Atraumatic clip	

Safe handling and preparation

CAUTION

Federal law restricts this device to sale by, or on order of a physician!

- ▶ Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- ▶ Read, follow, and keep the instructions for use.
- ▶ Use the product only in accordance with its intended use, see Intended use.
- ▶ Remove the transport packaging and clean the new product, either manually or mechanically, prior to its initial sterilization.
- ▶ Store any new or unused products in a dry, clean, and safe place.
- ▶ Prior to each use, inspect the product for loose, bent, broken, cracked, worn, or fractured components.
- ▶ Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- ▶ Replace any damaged components immediately with original spare parts.
- ▶ To avoid damage to the working end: Carefully insert the product through the working channel (e.g. trocar).

Safe operation

Risk of injury and/or malfunction!
▶ Always carry out a function check prior to using the product.

Risk of injury when using the product beyond the field of view!
▶ Apply the product only under visual control.

Clip applying/removal forceps PL530R

- ▶ Pick up the clips with the nibs 4 of the clip applying/removal forceps in the holes 5, see Fig. 1.

Clip applying/removal forceps PL531R

Note
Clip applying/removal forceps PL531R are only suitable for placing the clip in an axial direction.

- ▶ Insert the clips into the jaw pieces 1 of the clip applying/removal forceps as far as they will go, see Fig. 2.

Validated reprocessing procedure

General safety instructions

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for sterile processing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

Note

Mechanical reprocessing should be favored over manual cleaning as it gives better and more reliable results.

Note

Successful processing of this medical device can only be ensured if the processing method is first validated. The operator/sterile processing technician is responsible for this. The recommended chemistry was used for validation.

Note

If there is no final sterilization, then a virucidal disinfectant must be used.

Note

For the latest information on reprocessing and material compatibility see also the Aesculap extranet at www.extranet.bbraun.com

The validated steam sterilization procedure was carried out in the Aesculap sterile container system.

General information

Dried or affixed surgical residues can make cleaning more difficult or ineffective and lead to corrosion. Therefore the time interval between application and processing should not exceed 6 h; also, neither fixating pre-cleaning temperatures >45 °C nor fixating disinfecting agents (active ingredient: aldehydes/alcohols) should be used.

Excessive measures of neutralizing agents or basic cleaners may result in a chemical attack and/or to fading and the laser marking becoming unreadable visually or by machine for stainless steel.

Residues containing chlorine or chlorides e.g. in surgical residues, medicines, saline solutions and in the service water used for cleaning, disinfection and sterilization will cause corrosion damage (pitting, stress corrosion) and result in the destruction of stainless steel products. These must be removed by rinsing thoroughly with demineralized water and then drying.

Additional drying, if necessary.

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- Optical changes of materials, e.g. fading or discoloration of titanium or aluminum. For aluminum, the application/process solution only needs to be of pH >8 to cause visible surface changes.
- Material damage such as corrosion, cracks, fracturing, premature aging or swelling.
- ▶ Do not use metal cleaning brushes or other abrasives that would damage the product surfaces and could cause corrosion.
- ▶ Further detailed advice on hygienically safe and material-/value-preserving reprocessing can be found at www.a-k-i.org, link to Publications, Red Brochure – Proper maintenance of instruments.

Disassembling the product before carrying out the reprocessing procedure

- ▶ Remove the sealing cap from the Luer lock connector.

Preparations at the place of use

- ▶ If applicable, rinse non-visible surfaces preferably with deionized water, with a disposable syringe for example.
- ▶ Remove any visible surgical residues to the extent possible with a damp, lint-free cloth.
- ▶ Transport the dry product in a sealed waste container for cleaning and disinfection within 6 hours.

Cleaning/disinfection

Product-specific safety notes on the reprocessing procedure

Damage to the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures!

- ▶ Use cleaning and disinfecting agents approved for plastics and high-grade steel according to manufacturer's instructions.
- ▶ Observe specifications regarding concentration, temperature and exposure time.
- ▶ Do not exceed the maximum allowable temperature of 55 °C.

- ▶ Carry out ultrasound cleaning:
 - as an effective mechanical supplement to manual cleaning/disinfecting.
 - as a pre-cleaning procedure for products with encrusted residues, in preparation for mechanical cleaning/disinfecting.
 - as an integrated mechanical support measure for mechanical cleaning/disinfecting.
 - for additional cleaning of products with residues left after mechanical cleaning/disinfecting.

General cleaning instructions for clips

- ▶ Products that need to be opened via a joint, hinge etc., must be cleaned in open position.
- ▶ Products that do not need to be opened via a joint, hinge etc., must be opened using an applying forceps, wire or similar object (e.g. device) for cleaning/precleaning.
- ▶ Ensure that the wire or similar object is thick enough to keep the clips half open.
- ▶ To avoid rinsing blind spots, a wire or similar object must be positioned in the area of the olive or in the non-serrated area.
- ▶ Position the product in the tray in such a way that the best possible cleaning effect will be achieved.

Note

The similar object must be made of metal and if possible be of round cross-section. To avoid rinsing blind spots, do not use plastic materials or elastic tubes (silicone or PVC).

The applying forceps can also be used for the manual pre-cleaning of the serrated area.

Validated cleaning and disinfection procedure

Validated procedure	Specific requirements	Reference
Manual cleaning with immersion disinfection <ul style="list-style-type: none"> Clips 	<ul style="list-style-type: none"> Suitable cleaning brush 20 ml disposable syringe Keep working ends open for cleaning. When cleaning products with movable hinges, ensure that these are in an open position and, if applicable, move the joint while cleaning. Drying phase: Use a lint-free cloth or medical compressed air 	Chapter Manual cleaning/disinfection and subchapter: <ul style="list-style-type: none"> Chapter Manual cleaning with immersion disinfection
Manual cleaning with ultrasound and immersion disinfection <ul style="list-style-type: none"> Clip applying/removal forceps 	<ul style="list-style-type: none"> Suitable cleaning brush 20 ml disposable syringe Keep working ends open for cleaning. When cleaning instruments with movable hinges, ensure that these are in an open position and, if applicable, move the hinge while cleaning. Drying phase: Use lint-free cloth or medical compressed air 	Chapter Manual cleaning/disinfection and sub-chapter: <ul style="list-style-type: none"> Chapter Manual cleaning with ultrasound and immersion disinfection
Manual pre-cleaning with brush and subsequent mechanical alkaline cleaning and thermal disinfection <ul style="list-style-type: none"> Clip applying/removal forceps 	<ul style="list-style-type: none"> Suitable cleaning brush 20 ml disposable syringe Place the product in a tray that is suitable for cleaning (avoiding rinsing blind spots). Connect components with lumens and channels directly to the rinsing port of the injector carriage. Keep working ends open for cleaning. Place products in the tray with their hinges open. 	Chapter Mechanical cleaning/disinfection with manual pre-cleaning and subchapter: <ul style="list-style-type: none"> Chapter Manual pre-cleaning with a brush Chapter Mechanical alkaline cleaning and thermal disinfection
Manual pre-cleaning with ultrasound and brush, and subsequent mechanical alkaline cleaning and thermal disinfection <ul style="list-style-type: none"> Clips 	<ul style="list-style-type: none"> Suitable cleaning brush 20 ml disposable syringe Place the instrument in a tray that is suitable for cleaning (avoiding rinsing blind spots). Connect components with lumens and channels directly to the rinsing port of the injector carriage. Keep working ends open for cleaning. Place instruments in the tray with their hinges open. 	Chapter Mechanical cleaning/disinfection with manual pre-cleaning and sub-chapter: <ul style="list-style-type: none"> Chapter Manual pre-cleaning with ultrasound and brush Chapter Mechanical alkaline cleaning and thermal disinfection

Manual cleaning/disinfection

- Prior to manual disinfecting, allow water to drip off for a sufficient length of time to prevent dilution of the disinfecting solution.
- After manual cleaning/disinfection, check visible surfaces visually for residues.
- Repeat the cleaning/disinfection process if necessary.

Manual cleaning with immersion disinfection

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Disinfecting cleaning	RT (cold)	>15	2	D-W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
II	Intermediate rinse	RT (cold)	1	-	D-W	-
III	Disinfection	RT (cold)	15	2	D-W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
IV	Final rinse	RT (cold)	1	-	FD-W	-
V	Drying	RT	-	-	-	-

D-W: Drinking water
 FD-W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)
 RT: Room temperature
 *Recommended: BBraun Stabimed

- Note the information on appropriate cleaning brushes and disposable syringes, see Validated cleaning and disinfection procedure.

Phase I

- Fully immerse the product in the cleaning/disinfectant for at least 15 min. Ensure that all accessible surfaces are moistened.
- Clean the product with a suitable cleaning brush in the solution until all discernible residues have been removed from the surface.
- If applicable, brush through non-visible surfaces with an appropriate cleaning brush for at least 1 min.
- Mobilize non-rigid components, such as set screws, links, etc. during cleaning.
- Thoroughly rinse through these components with the cleaning disinfectant solution (at least five times), using a disposable syringe.

Phase II

- Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- Drain any remaining water fully.

Phase III

- Fully immerse the product in the disinfectant solution.
- Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- Rinse lumens at least 5 times at the beginning of the exposure time using an appropriate disposable syringe. Ensure that all accessible surfaces are moistened.

Phase IV

- Rinse/flush the product thoroughly (all accessible surfaces).
- Mobilize non-rigid components, such as set screws, joints, etc. during final rinse.
- Rinse lumens with an appropriate disposable syringe at least five times.
- Drain any remaining water fully.

Phase V

- Dry the product in the drying phase with suitable equipment (e.g. cloth, compressed air), see Validated cleaning and disinfection procedure.

Manual cleaning with ultrasound and immersion disinfection

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Ultrasonic cleaning	RT (cold)	>15	2	D-W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
II	Intermediate rinse	RT (cold)	1	-	D-W	-
III	Disinfection	RT (cold)	15	2	D-W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
IV	Final rinse	RT (cold)	1	-	FD-W	-
V	Drying	RT	-	-	-	-

D-W: Drinking water
 FD-W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)
 RT: Room temperature
 *Recommended: BBraun Stabimed

- Note the information on appropriate cleaning brushes and disposable syringes, see Validated cleaning and disinfection procedure.

Phase I

- Clean the product in an ultrasonic cleaning bath (frequency 35 kHz) for at least 15 min. Ensure that all accessible surfaces are immersed and acoustic shadows are avoided.
- Clean the product with a suitable cleaning brush in the solution until all discernible residues have been removed from the surface.
- If applicable, brush through non-visible surfaces with an appropriate cleaning brush for at least 1 min.
- Mobilize non-rigid components, such as set screws, links, etc. during cleaning.
- Thoroughly rinse through these components with the cleaning disinfectant solution (at least five times), using a disposable syringe.

Phase II

- Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- Drain any remaining water fully.

Phase III

- Fully immerse the product in the disinfectant solution.
- Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- Rinse lumens at least five times at the beginning of the exposure time with an appropriate disposable syringe. Ensure that all accessible surfaces are moistened.

Phase IV

- Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as set screws, joints, etc. during final rinse.
- Rinse lumens with an appropriate disposable syringe at least five times.
- Drain any remaining water fully.

Phase V

- Dry the product in the drying phase with suitable equipment (e.g. cloth, compressed air), see Validated cleaning and disinfection procedure.

Mechanical cleaning/disinfection with manual pre-cleaning

Note

The cleaning and disinfection device must be of tested and approved effectiveness (e.g. FDA approval or CE mark according to DIN EN ISO 15883).

Note

The cleaning and disinfection device used for processing must be serviced and checked at regular intervals.

Manual pre-cleaning with a brush

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Disinfectant cleaning	RT (cold)	>15	2	D-W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
II	Rinsing	RT (cold)	1	-	D-W	-

D-W: Drinking water
 RT: Room temperature
 *Recommended: BBraun Stabimed

- Note the information on appropriate cleaning brushes and disposable syringes, see Validated cleaning and disinfection procedure.

Phase I

- ▶ Fully immerse the product in the cleaning/disinfectant for at least 15 min. Ensure that all accessible surfaces are moistened.
- ▶ Clean the product with a suitable cleaning brush in the solution until all discernible residues have been removed from the surface.
- ▶ If applicable, brush through non-visible surfaces with an appropriate cleaning brush for at least 1 min.
- ▶ Mobilize non-rigid components, such as set screws, links, etc. during cleaning.
- ▶ Thoroughly rinse through these components with the cleaning disinfectant solution (at least five times), using a disposable syringe.

Phase II

- ▶ Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- ▶ Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.

Manual pre-cleaning with ultrasound and brush

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Ultrasonic cleaning	RT (cold)	>15	2	D-W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
II	Rinsing	RT (cold)	1	-	D-W	-

D-W: Drinking water
RT: Room temperature
*Recommended: BBraun Stabimed

- ▶ Note the information on appropriate cleaning brushes and disposable syringes, see Validated cleaning and disinfection procedure.

Phase I

- ▶ Clean the product in an ultrasonic cleaning bath (frequency 35 kHz) for at least 15 min. Ensure that all accessible surfaces are immersed and acoustic shadows are avoided.
- ▶ Clean the product with a suitable cleaning brush in the solution until all discernible residues have been removed from the surface.
- ▶ If applicable, brush through non-visible surfaces with an appropriate cleaning brush for at least 1 min.
- ▶ Mobilize non-rigid components, such as set screws, links, etc. during cleaning.
- ▶ Thoroughly rinse through these components with the cleaning disinfectant solution (at least five times), using a disposable syringe.

Phase II

- ▶ Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- ▶ Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.

Mechanical alkaline cleaning and thermal disinfecting

Machine type: single-chamber cleaning/disinfection device without ultrasound

Phase	Step	T [°C/°F]	t [min]	Water quality	Chemical
I	Pre-rinse	<25/77	3	D-W	-
II	Cleaning	55/131	10	FD-W	<ul style="list-style-type: none">■ Concentrate, alkaline:<ul style="list-style-type: none">- pH = 13- <5 % anionic surfactant■ 0.5 % working solution<ul style="list-style-type: none">- pH = 11*
III	Intermediate rinse	>10/50	1	FD-W	-
IV	Thermal disinfecting	90/194	5	FD-W	-
V	Drying	-	-	-	According to the program for cleaning and disinfection device

D-W: Drinking water
FD-W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)
*Recommended: BBraun Helimatic Cleaner alkaline

- ▶ Check visible surfaces for residues after mechanical cleaning/disinfecting.

Inspection, maintenance and checks

CAUTION

Damage (metal seizure/friction corrosion) to the product caused by insufficient lubrication!

- ▶ Prior to function checks, lubricate moving parts (e.g. joints, pusher components and threaded rods) with maintenance oil suitable for the respective sterilization process (e.g. for steam sterilization: Aesculap STERILIT® I oil spray JG600 or STERILIT® I drip lubricator JG598).

- ▶ Allow the product to cool down to room temperature.
- ▶ After each complete cleaning, disinfecting and drying cycle, check that the product is dry, clean, operational, and free of damage (e.g. broken insulation or corroded, loose, bent, broken, cracked, worn, or fractured components).
- ▶ Dry the product if it is wet or damp.
- ▶ Repeat cleaning and disinfection of products that still show impurities or contamination.
- ▶ Check that the product functions correctly.
- ▶ Immediately put aside damaged or inoperative products and send them to Aesculap Technical Service, see Technical Service.
- ▶ Check for compatibility with associated products.

Packaging

- ▶ Appropriately protect products with fine working tips.
- ▶ Place the product in its holder or on a suitable tray.
- ▶ Pack trays appropriately for the intended sterilization process (e.g. in Aesculap sterile containers).
- ▶ Ensure that the packaging provides sufficient protection against recontamination of the product during storage.

Steam sterilization

- ▶ Check to ensure that the sterilizing agent will come into contact with all external and internal surfaces (e.g. by opening any valves and faucets).
- ▶ Validated sterilization process
 - Steam sterilization using fractionated vacuum process
 - Steam sterilizer according to DIN EN 285 and validated according to DIN EN ISO 17665
 - Sterilization using fractionated vacuum process at 134 °C/holding time 5 min
- ▶ When sterilizing several products at the same time in a steam sterilizer, ensure that the maximum load capacity of the steam sterilizer specified by the manufacturer is not exceeded.

Sterilization for the US market

- Aesculap advises against sterilizing the device by flash sterilization or chemical sterilization.
 - Sterilization may be accomplished by a standard prevacuum cycle in a steam autoclave.
- To achieve a sterility assurance level of 10⁻⁶, Aesculap recommends the following parameters:

Aesculap Orga Tray/Sterile container (perforated bottom) Minimum cycle parameters*			
Sterilization method	Temp.	Time	Minimum drying time
Prevacuum	270 °F/275 °F	4 min	20 min

*Aesculap has validated the above sterilization cycle and has the data on file. The validation was accomplished in an Aesculap sterile container cleared by FDA for the sterilization and storage of these products. Other sterilization cycles may also be suitable, however individuals or hospitals not using the recommended method are advised to validate any alternative method using appropriate laboratory techniques. Use an FDA cleared accessory to maintain sterility after processing, such as a wrap, pouch, etc.

Storage

- ▶ Store sterile products in germ-proof packaging, protected from dust, in a dry, dark, temperature-controlled area.

Technical Service

WARNING

Risk of injury and/or malfunction!

- ▶ Do not modify the product.

- ▶ For service and repairs, please contact your national B. Braun/Aesculap agency.

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 (7461) 95-1602
Fax: +49 (7461) 16-5621
E-Mail: ats@aesculap.de

Or in the US:

Aesculap Inc.
Attn. Aesculap Technical Services
615 Lambert Pointe Drive
Hazelwood
MO, 63042
Aesculap Repair Hotline
Phone: +1 (800) 214-3392
Fax: +1 (314) 895-4420

Other service addresses can be obtained from the address indicated above.

Disposal

- ▶ Adhere to national regulations when disposing of or recycling the product, its components and its packaging!

Distributor in the US/Contact in Canada for product information and complaints

3773 Corporate Parkway
Center Valley, PA, 18034,
USA

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